



SUGGESTED CURING TIMES

DENTAL

	STANDARD	HIGH POWER	XTRA POWER
POWER (mW/cm²)	1000* ●	1400* ●	3200' ☀
POWER BUTTON TIMING INTERVAL LIGHTS MODE/STATUS LIGHT TIME/MODE CHANGE BUTTON	 5 sec. 10 sec. 15 sec. 20 sec.	 1 sec. 2 sec. 3 sec. 4 sec.	 3 sec.
PER LAYER	1 X 10 SECONDS	2 X 4 SECONDS	1 X 3 SECONDS
FINAL CURE	1 X 20 SECONDS	3 X 4 SECONDS	2 X 3 SECONDS

ORTHO

METAL & CERAMIC BRACKETS	1 X 20 SECONDS	3 X 4 SECONDS	2 X 3 SECONDS
-------------------------------------	----------------	---------------	---------------

Applicable for all Ultradent light cured materials, current and future, including: Vit-I-escence, Vit-I-escence Colors, Amelogen Plus, PermaFlo, PermaFlo Pink, PermaFlo Purple, PermaFlo DC, UltraSeal XT plus, Ultra-Blend plus, and PermaSeal.

Applicable for all Ultradent adhesive materials, current and future, including: Peak Universal Bond, Peak LC Bond, Peak SE, and PQ1. OpalDam should only be cured with standard mode.

Applicable for all Opal Orthodontics light cured materials, current and future, including: Opal Bond, Opal Bond Flow, Opal Bond MV, and Opal Seal.

*As measured by a properly calibrated Demitron Light Meter

†As measured by an Integrating Sphere

ULTRADENT
PRODUCTS, INC.

800.552.5512 WWW.ULTRADENT.COM

